

Infinity



USER MANUAL

v1.4

Table of contents

1	Introduction.....	3
2	Installation, safety and maintenance.....	3
	Box contents.....	3
	Installation.....	3
	Instructions.....	4
	Use.....	4
3	Controls.....	5
	On / Off Switch.....	5
	Zooming.....	5
	Viewing Modes.....	6
	Blocking the autofocus.....	6
	Reading aid.....	7
	Brightness.....	7
	Tableau.....	7
	Brake Handles.....	8
4	General Information.....	8
	Discarding the Infinity.....	8
5	Technical Specifications.....	9
6	Control Overview.....	10

1 Introduction

The Infinity CCTV is well known for its easy to use features and an excellent image quality. The Infinity is delivered with an autofocus system and an ideal reading table.


2 Installation, safety and maintenance

Box contents

The Infinity box contains the following:

- Infinity
- TFT screen
- Power cable
- User manual

Installation

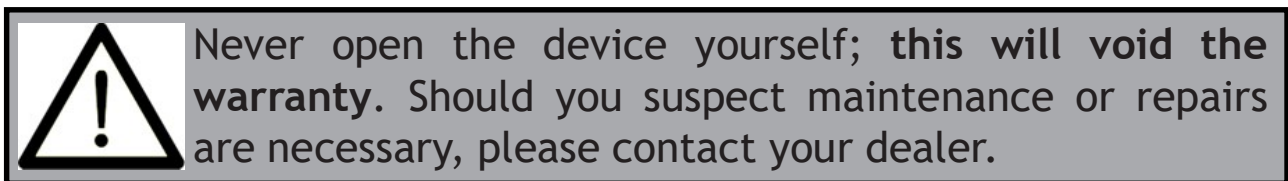


Please read the following instructions carefully before you install and use the CCTV.

If you are having difficulties installing the CCTV, or if you still have questions after reading the user manual, please contact the dealer.

Instructions

- Only use your CCTV indoors. Do not expose the device to extreme temperature fluctuations or humidity. Do not place the Infinity directly next to a heat source or in direct sunlight.
- Ensure the CCTV is in a well ventilated room and that the ventilation openings are open.
- Place the CCTV on a stable surface to prevent it from falling. Ensure that the Infinity is placed 10cm from the edge of the table.

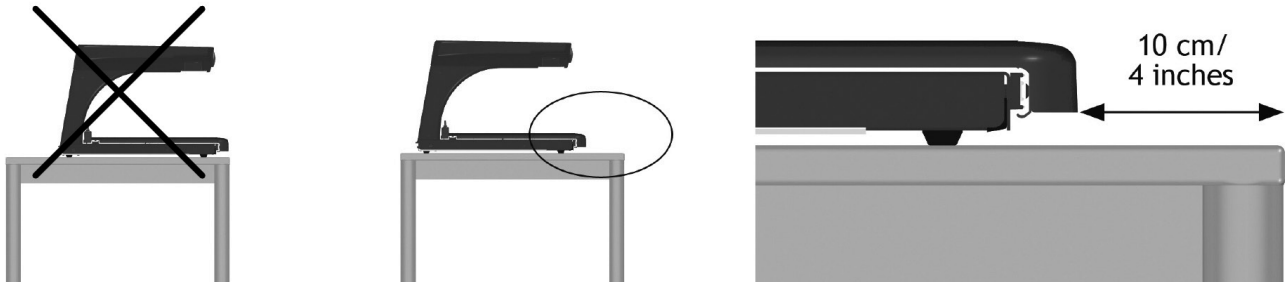

- Should you want to place a different monitor on the Infinity, this monitor may not exceed 25kg.
- Use only the adapter that is included and connect the power cable on an earthed socket. Make sure that the cables cannot be damaged.
- A monitor can be sensitive to strong magnetic fields. Please ensure that speakers of audio equipment are not too close to the monitor.

Use

- Please avoid direct sunlight on the screen and reading table.
- Be very careful with the moving parts of the tableau and the serrated metal edge on the left side of the tableau
- The Infinity requires little or no maintenance. If you want to clean the device on the outside, please use only a soft dry cloth. The use of aggressive cleaning products like alcohol, acetone and other chemical products are prohibited.
- Please do not place anything but a screen on top of the Infinity e.g. no books, plants, clocks, etc.
- Do not use the Infinity as a support to walk, sit or stand on.
- Switch off the device when you're not using it.

3 Controls

See chapter 6 for an overview of the symbols used in the descriptions. This manual will explain the controls for both push and turning knobs. Please check your device to see which description applies to your device.



On / Off switch

The on / off switch is located on the right side of the device. A TFT monitor might have its own power switch.



Zooming

Press the up button multiple times to zoom in. Press the down button multiple times to zoom out.



Viewing modes

This button allows you to select different viewing modes. There are several high contrast modes available - please refer to the specifications in the back of the manual for all available modes. The last used mode will be stored in memory and automatically selected upon power up.

By pressing the button briefly you can select the available photo and high contrast modes. After a series of combinations the photo mode will automatically be selected.



Blocking the autofocus

The Infinity will always start with the continuous Autofocus mode. In some circumstances (i.e. writing) it may be necessary to disable the continuous autofocus and enable the Fixed Focus.

Use the zooming controls to block the autofocus.

Press both zooming buttons simultaneously to block the autofocus. Press either zoom button to release the fixed focus.



Reading aid

To activate the reading line aid, keep the brightness button pressed and then press the zoom up or zoom down button. Press the zoom up or zoom down button multiple times to position the reading line. To remove the reading line from the screen, keep the brightness button pressed and then press either the zoom up or zoom down button.



Brightness


Press the button to adjust the brightness of the screen.



Tableau

The Infinity has a smooth moving X Y tableau. You can place a book, photo, magazine etc. to enlarge it on the screen. The tableau can be moved from right to left or left to right (X) or forwards and backwards (Y).

Tip: Place your book at the back of the tableau against the upstanding edge. By doing so the text will read straight and the reading line or block blanking will run parallel to your text as well.

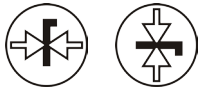

Brake handles

Underneath the tableau you will find two brake handles with which you can restrict the movement of the tableau.

The left brake handle controls the movement from left to right.

The right handle controls the movement from up to down.

The handles have 3 positions:

left = free movement of the tableau


middle = free movement, with slight resistance

right = all movement is blocked

4 General Information

This product is tested for safety and electromagnetic interference and meets the requirements of both the Low Voltage Directive and the EMC Directive as laid out by the European Parliament and the Council of the European Union.

Discarding the Infinity

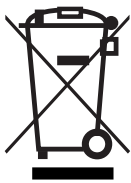

This symbol of the crossed out wheelie bin indicates that the product must not be disposed of with your other household waste. Check local regulations for proper disposal of electronic products.

5 Technical specifications

Viewing Modes	Photo	Colours
	High Contrast	Black - White
		White - Black
		Black - Yellow
		Yellow - Black
		Yellow - Blue
		Blue - Yellow
		Black - Green
Green - Black		
Zooming	Minimum - Maximum	3.5x - 54x (22" TFT)
Focus		Autofocus (incl. blocking option)
Reading aid		Reading line horizontal
Dimensions	Infinity Basic with TFT	56 x 52 x 43 cm
Weight	22"	16.5 Kg
Power SUPply	Infinity Basic	12v DC / 2A
	TFT Monitor	100 - 240VAC / 0,8A
Conditions of use		Temperature +5°C to +50°C
		Humidity <70%, no condensation
Transport and storage conditions		Temperature +0°C to +50°C
		Humidity <90%, no condensation

Should you have any questions after reading this manual, please contact:

Rehan Electronics Ltd.
Aughrim Rd., Carnew
Wicklow Y14 NY60
Rep. of Ireland
Tel.: 00353 (0) 53 9426742
Email: sales@rehanelectronics.ie
Web.: www.rehanelectronics.ie

6 Control overview



