
# i-See HD

GEBRUIKSAANWIJZING  
USER MANUAL  
GEBRAUCHSANLEITUNG  
ISTRUZIONI D'USO  
MANUAL DE INTRUCCIONES  
MANUEL DE L'UTILISATEUR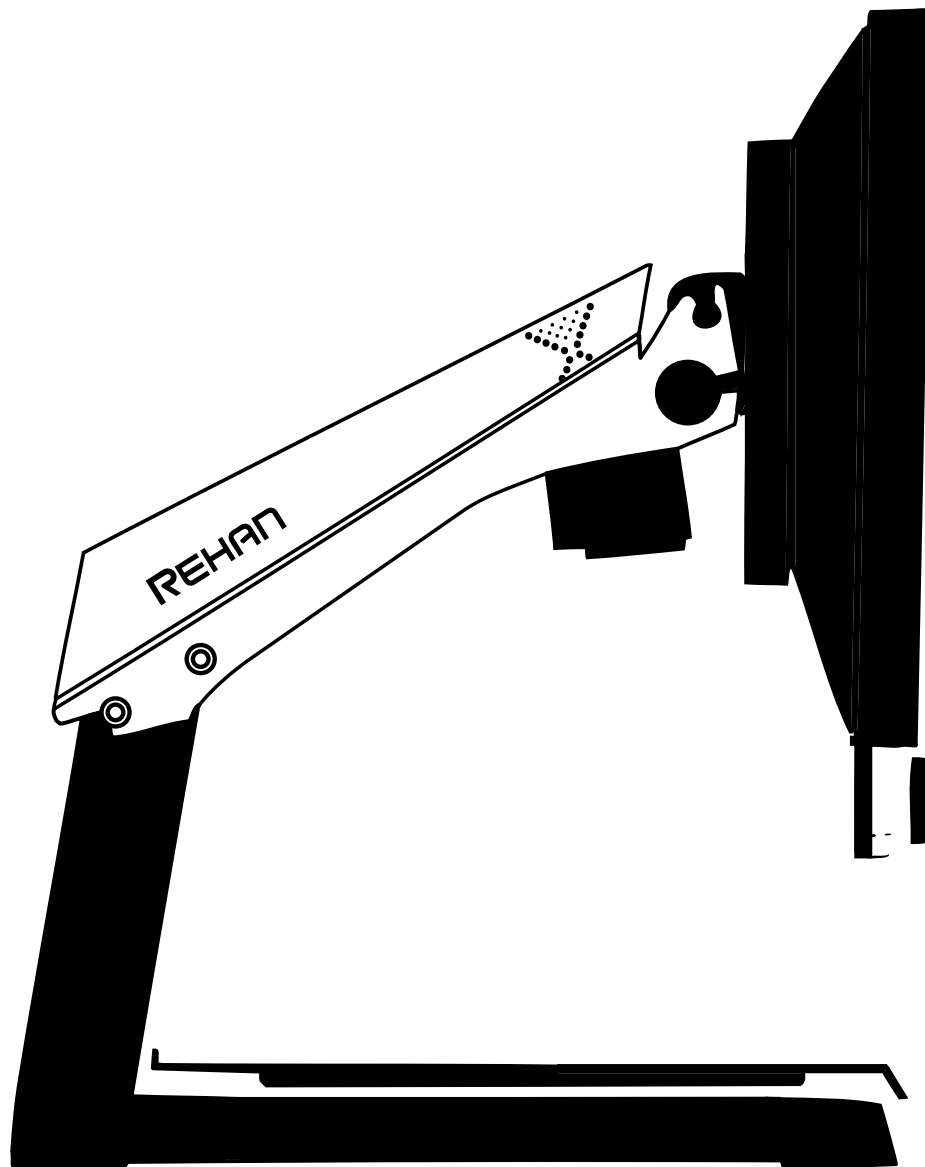

**Nederlands ..... 5**

NL

**English..... 13**

EN

**Deutsch ..... 21**

DE

**Italiano ..... 29**

IT

**Español ..... 37**

ES

**Français ..... 45**

FR

---

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1	Introductie .....	6
2	Inhoud verpakking .....	6
3	Installatie .....	6
3.1	TFT scherm bevestigen .....	6
3.2	Scherm kantelen .....	7
3.3	Scherm in hoogte verstelbaar .....	7
3.4	Kabels bevestigen .....	8
4	Gebruik .....	8
5	Bediening .....	9
5.1	Aan-/uitschakelaar .....	9
5.2	Beeldweergave .....	10
5.3	Vergroten - verkleinen .....	10
5.4	Voorkeursvergroting .....	10
5.5	Autofocusblokkering .....	10
5.6	Helderheid .....	10
5.7	Leeshulp .....	10
5.8	Tableau .....	11
5.9	Remhendel .....	11
6	Richtlijnen .....	11
7	De i-See HD recyclen .....	11
8	Technische specificaties .....	12

---

# 1 Introductie

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---

# 1 Introduction

Congratulations with your i-See HD; a desktop video magnifier of high quality that is especially user friendly.

Read this manual thoroughly before you start to use your i-See HD. This manual contains instructions on how the device needs to be installed, and describes all of the functions of the i-See HD.

EN

## 2 Package contents

The package of the i-See HD contains the following:

- i-See HD base
- TFT screen for with control interface
- Powercable
- Manual

## 3 Installation


Never open up the device yourself; **this will void your warranty.** If you suspect the device is in need of service or repairs, please contact your supplier.

The i-See HD will be delivered to you in a pre-assembled state. All you have to do is connect the TFT monitor to the base, attach the cables to the control interface and TFT monitor and connect the power cable to the base.

### 3.1 Attach TFT screen


You can attach the TFT monitor by hanging it on the base and tightening the bolts on both rear sides of the screen.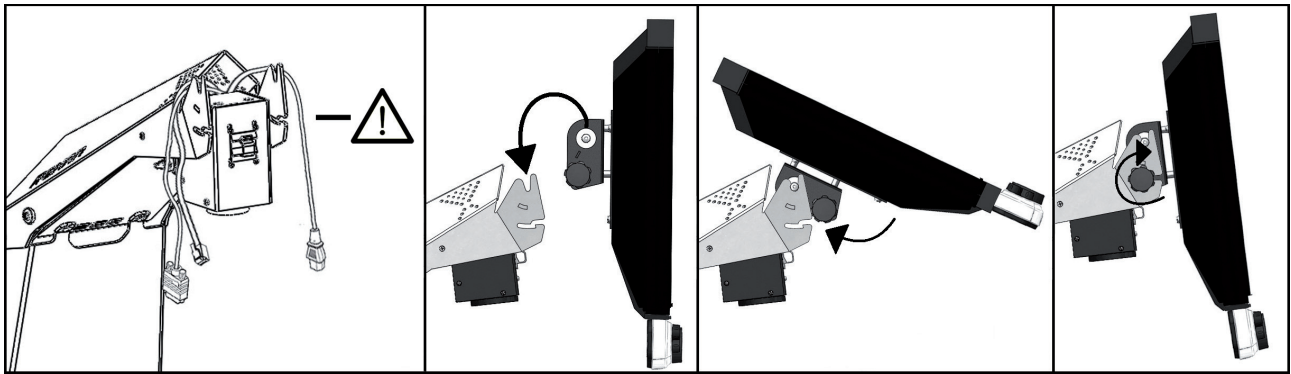

### 3.2 Tilt the screen


Loosen the bolts at both rear sides of the screen; this loosens up the screen so you can move it in your preferred position. Tighten the bolts to secure the screen at that position.

EN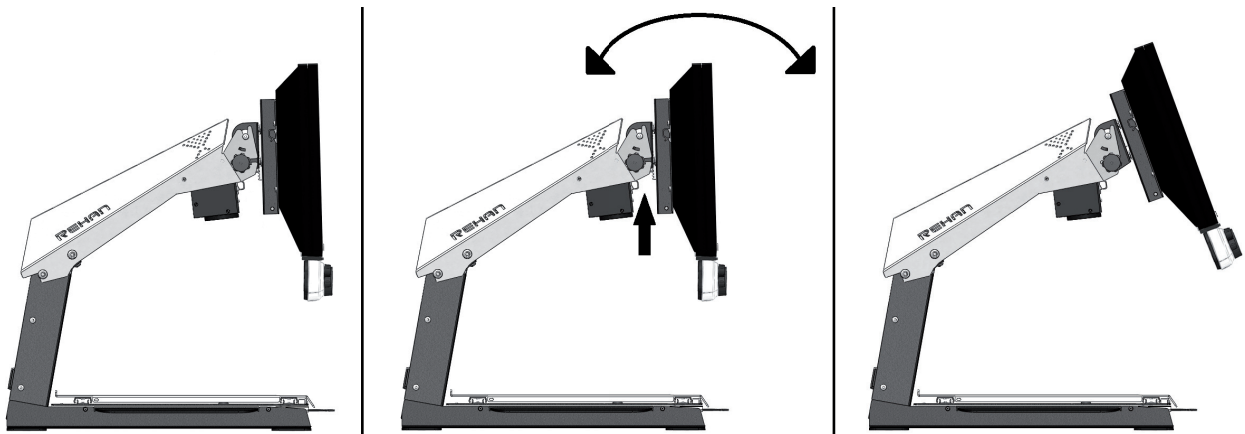


### 3.3 Adjust the height of the screen

With the i-See HD it is possible to adjust the height of the screen. Simply grab the lever at the right side of the TFT screen, pull it towards you and, while holding your left hand at the lower left corner of the screen, slide the screen up or down. Release the lever to secure the screen at that position.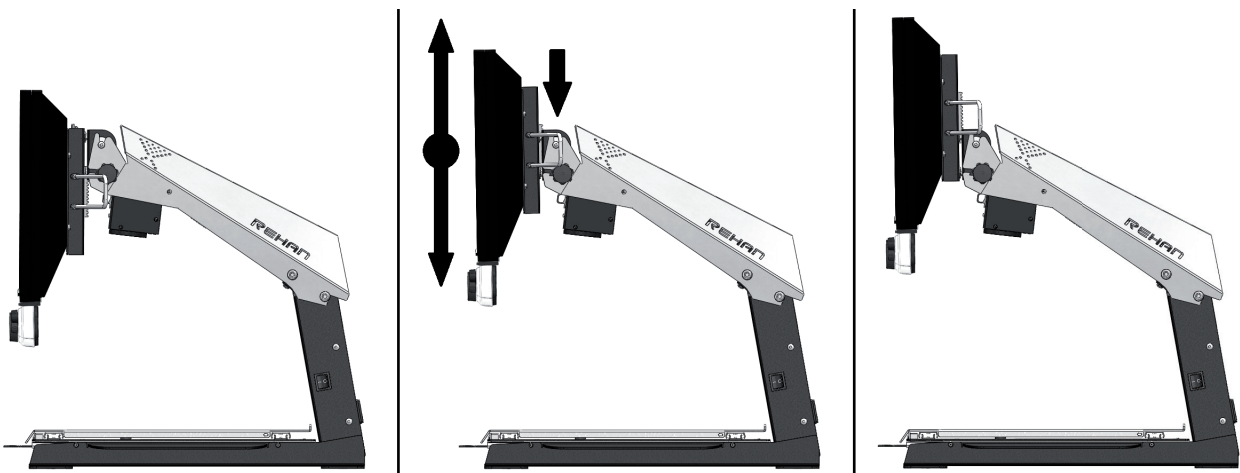


### 3.4 Connecting the cables

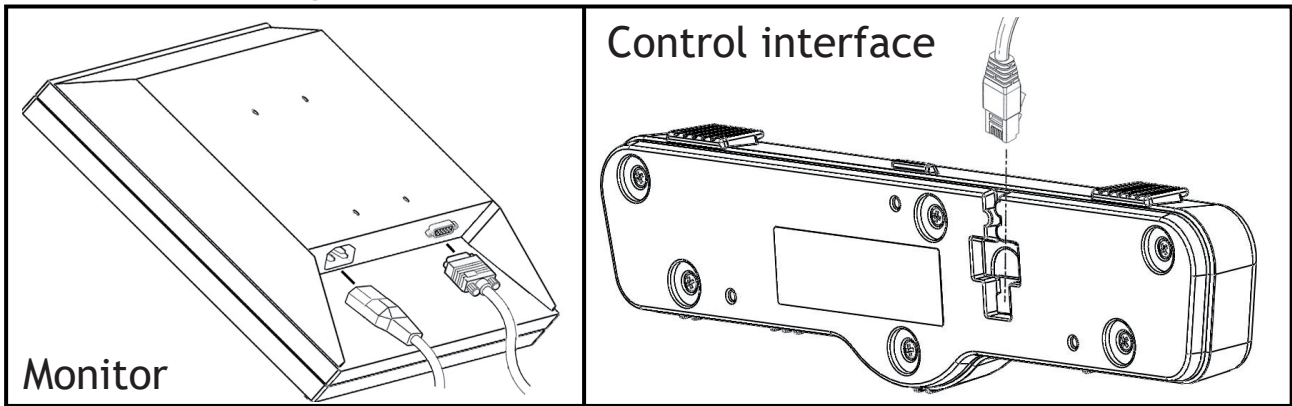EN

## 4 Use

- Avoid direct sunlight on the screen and tableau.
- Be careful when handling the moving parts of the reading table.
- The i-See HD requires little or no maintenance. If you want to clean the exterior of the device, only use a dry cloth. The use of aggressive cleaning products like alcohol, acetone or other chemical substances is prohibited.
- Do not use the i-See HD as a support for walking, sitting down or getting up.
- Switch off the device if you do not intend to use it.
- Ensure that the i-See HD is placed in a well ventilated room.
- Place the i-See HD on a stable surface to prevent it from falling.
- Only use your i-See HD indoors. Do not expose the i-See HD to extreme temperature fluctuations or humidity. Do not place your i-See HD next to a heat source or in direct sunlight.
- Only use the supplied cables and connect the power cable to an earthed socket.
- Ensure that the i-See HD is placed at a minimal distance of 10 cm / 4 inches from the edge of the table.

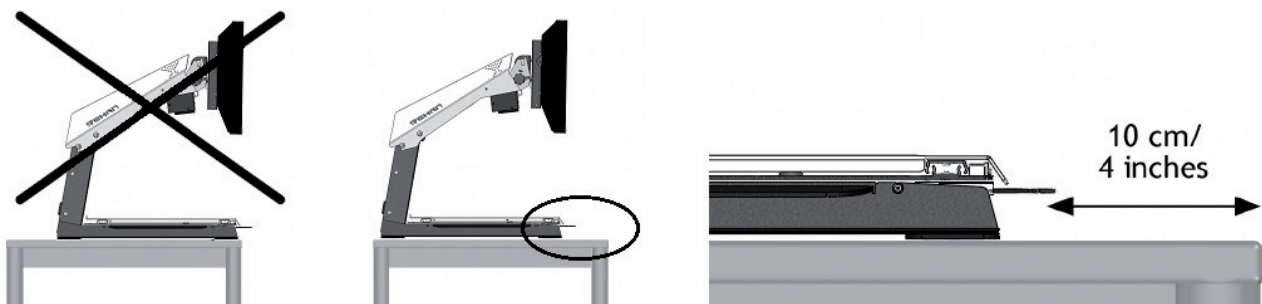
---

## 5 Controls

### 5.1 On/off switch

The on/off switch of the i-See HD is located on the right hand side of the pillar.

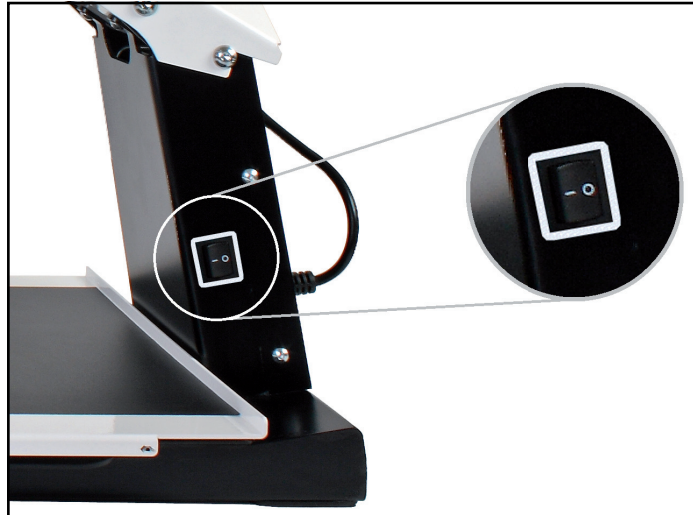
EN

The control interface contains all the controls of the i-See HD.

### 5.2 Display modes

Push the knob to switch between photograph modes or high contrast modes. The last used mode will automatically be selected. Turn the knob to select different display modes.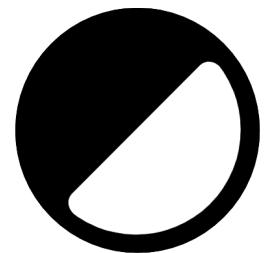


You can find a detailed overview of the available display modes in the technical specifications section of this manual.

---

### 5.3 Zooming in and out

Turn the knob to the right to zoom in. Turn the knob to the left to zoom out.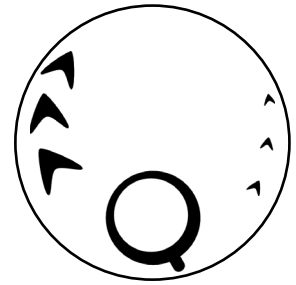

### 5.4 Zoom presets

While reading texts and looking at images, you require a zoom setting that suits your needs. The i-See HD contains a function that memorizes these preferred zoom settings for you.

This means that when you switch from the photograph mode to a text mode, the i-See HD will automatically switch to the last zoom setting you used in the text mode. The same goes when you switch from a text mode to the photograph mode - the device will automatically switch to the last zoom setting you used while viewing photos.


### 5.5 Blocking the autofocus

The i-See HD will always automatically focus on the object on the tableau. However, in certain situations (like during writing), it can be useful to block the autofocus.

Push the zoom knob to block the autofocus. Turn the knob in either direction to unblock the autofocus.

### 5.6 Brightness

Turn the knob to the left to decrease or to the right to increase the brightness of the image on your screen.

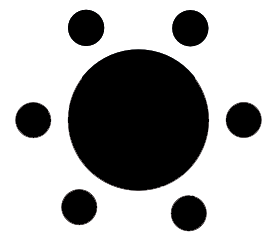### 5.7 Readings aid

By pushing the knob the user can choose between 4 different reading aids: a horizontal and vertical reading line, a horizontal and a vertical block blank. By subsequently turning the knob you can fine tune your chosen reading aid.

By keeping the knob pressed for 2 seconds, you can immediately switch your chosen reading aid on or off.

---

## 5.8 Tableau

The i-See HD contains a tableau on which you can place your reading material. The tableau can move from left to right (X-direction) and from front to back (Y-direction).

## 5.9 Brake handle

Underneath the tableau of the i-See HD you will find a single brake handle with which you can control the movability of the tableau. The handle has 3 positions:


Left = all movement of the tableau is blocked  
Middle = the movement from front to back (Y-direction) is blocked  
Right = the tableau can be moved freely

## 6 Regulations

The i-See HD is tested for safety and electromagnetic interference and meets the requirements of both the Low Voltage Directive and the EMC Directive as laid out by the European Parliament and the Council of the European Union.

The device also meets the requirements of FCC 47 CFR Part15B Class B.

## 7 Discarding the i-See HD

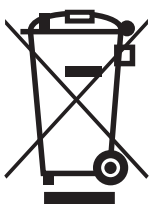This symbol of the crossed out wheelie bin indicates that the product must not be disposed of with your other household waste. Check local regulations for proper disposal of electronic products.

## 8 Technical specifications

General specifications (for 22")	
Magnification	3x – 75x
Resolution	720p
Dimensions	51 × 48 × 64 cm (screen in highest position) 20 × 19 × 25.2 inch
Weight	15.8kg / 34.8lb
Power supply	110 – 240V @ 50/60Hz
Conditions of use	<b>Temperature</b> +5°C – +35°C +41°F – +95°F <b>humidity</b> <70% (no condensation)
Conditions of storage and transport	<b>Temperature</b> +0°C – +50°C +32°F – +122°F <b>Humidity</b> <90% (no condensation)

If you have any questions or queries after reading this manual, please contact your supplier.

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**Rehan Electronics Ltd.